

# ADAPTATION ASSEMBLY INSTRUCTIONS



The therapeutic tricycle for people with disabilities

For questions or to place an order, please contact:

Amtryke, LLC.  
(800) 838-1845 x114  
amtryke@ambucs.org  
www.ambucs.org

## 1400/1600 Series Seat Back Spacer Kit Part Number 30-13-1401

**What's inside:** (4)- M6 x 1 7/8 Hex Head Bolts (4)- 1 1/2" Black Plastic Spacers

**Tools Needed:**  
1- 6mm Allen Wrench

### Product Description:

The 1400/1600 Series Seat Back Spacer kit makes it possible to move the padded back forward so the riders back is firmly against the padded back.

### Assembly Instructions

- 1) Located on the backside of the seat back are four 6mm screws that hold the padded portion to the metal bracket.
- 2) Remove the four 6mm screws using the 6mm Allen Wrench.
- 3) Do not discard the four 6mm screws you just removed, save them to return the seat back to factory settings when the rider grows.
- 4) Slide one 6mm x 1 7/8" screw through one of the holes in the metal bracket, slide a spacer over it. Spacer should be located between the metal bracket and the back of the padded back.
- 5) Align the threaded hole that is located on the back of the padded back with the screw, lightly thread screw by hand two or three turns, just enough to hold it in place. NOTE: It is important to do only one at a time and not to tighten the screws up fully until all four are partially started.
- 6) Repeat the process above with the three remaining screws and spacers, until all four are in place.
- 7) Use the 6mm Allen Wrench and tighten the four screws until they are securely tightened.

**BEFORE**



**AFTER**



Revised: 4/16/2016

n:\amtryke\manuals and instructions\accessories\1400 series seat back spacer kit\1400 series seat back spacer kit.doc