



# ADAPTATION ASSEMBLY INSTRUCTIONS



**AmTryke**<sup>®</sup>

*The therapeutic tricycle for people with disabilities*

For questions or to place an order, please contact:  
Amtryke, LLC.  
(800) 838-1845 x114  
amtryke@ambucs.org  
www.ambucs.org

## Bucket Seat

Part No. 30-02-0325 – AM-vertical seat post; Part No. 30-02-0328 – 1400 slanted seat post

### Instructions

You need the following tools to assemble the bucket seat:

#### TOOLS REQUIRED:

- 3 mm and 4 mm wrench/socket or nut driver
- Phillips head screwdriver

**NOTE:** Before assembling the bucket seat, please read through the *Assembly Guide* and become familiar with the assembly process.

#### PACKAGE CONTENTS:

- Grey Plastic Bucket Seat
- Chest Strap
- Lap Straps (2)
- Black Seat Post (either straight for AM or slanted for 1400)
- 8 mm Allen Head Machine Screws (4)
- Large Flat washers (4)
- Small Flat Washers (2)
- 6 mm Allen Head Machine Screws (2)
- 6 mm nylock nuts (2)
- 5 mm Phillips washer head screws (2)



**NOTE:** AmTryke models come with two styles of red seat post brackets. Straight or vertical seat posts fit onto AM style AmTrykes. Slanted seat posts fit onto ProSeries models 1400. Make sure which style of seat post your AmTryke uses before ordering, assembly and use.

1. Thread the chest strap through the two vertical slots in the upper side of the seat. **Be sure** to thread so hook portion of strap faces **away** from the body.
2. The lap straps can be attached at two different locations, depending on your rider. Use the two 3 mm hex head machine screws with a washer and nylock nuts to attach straps in holes in the side of seat. OR, thread lap straps through slots at side of seat and use Phillips head screws to attach straps into brass inserts on the back of the seat. **BE CAREFUL TO INSTALL LAP STRAPS SO HOOK PORTION FACES AWAY FROM THE RIDER.** This is for comfort.

Chest strap is one piece



Lap Straps can be mounted at side or in back of seat.



**NOTE:** Check to be sure that the straps are properly threaded through slots in the seat. The hook portion of the straps should always face up so it will not rub against or contact the skin of the rider.

3. Place the black seat post onto seat back. Line the four (4) holes up with the large brass inserts. Use the 8 mm Allen Head Machine Screws with the Large Washers to attach it to the seat. This requires a 4 mm Allen head wrench. Tighten securely, but do not over torque the screws.



*Black seat post attaches to bottom of grey bucket seat.*

OPTIONAL –

The Grey Bucket Seat may also be used with the H-Harness parts. Be sure to order either:

Part No. 30-11-0336G (11.5-inch) or Part No. 30-11-0337G (8 inch) The “G” designates that the ends of the straps contain grommets for fastening with screws.

1. Thread the grommet end of the straps through the horizontal holes in the seat bottom and seat back.
2. Attach to the brass inserts using the 5 mm Phillips Washer Head screws.



Parts included in the Grey Bucket Seat. Parts Bag shown below.



Revised: 6/20/2016

n:\amtryke\manuals and instructions\accessories\bucket seat\bucket seat.docx