



ADAPTATION ASSEMBLY INSTRUCTIONS



AmTryke[®]

The therapeutic tricycle for people with disabilities

For questions or to place an order, please contact:

Amtryke, LLC.
(800) 838-1845 x114
amtryke@ambucs.org
www.ambucs.org

Foot Cup Assembly:


Part #: 30-31-0149S-Small; 30-31-0149M-Medium

Contents

Pair of Molded Foot Cups

2 ea. 3" wide adhesive backed hook material

- 1) When installing foot cups on the AmTryke models listed above - remove the heel strap from the pedal plate.
- 2) Peel off the paper backing on the 3" wide adhesive backed hook material.
- 3) Apply the 3" wide adhesive hook material to the metal foot plate. Be careful not to cover the slots at the edges of the foot plate.
- 4) Re-install the heel strap through the bottom of the metal footplate.
- 5) Thread the heel straps and toe straps through the foot cups, push the foot cups securely against the hook material surface on the metal foot plate. Note; Line up heel plate with inner slot of foot cup. Pull slack out of straps as you push firmly down on foot cup.

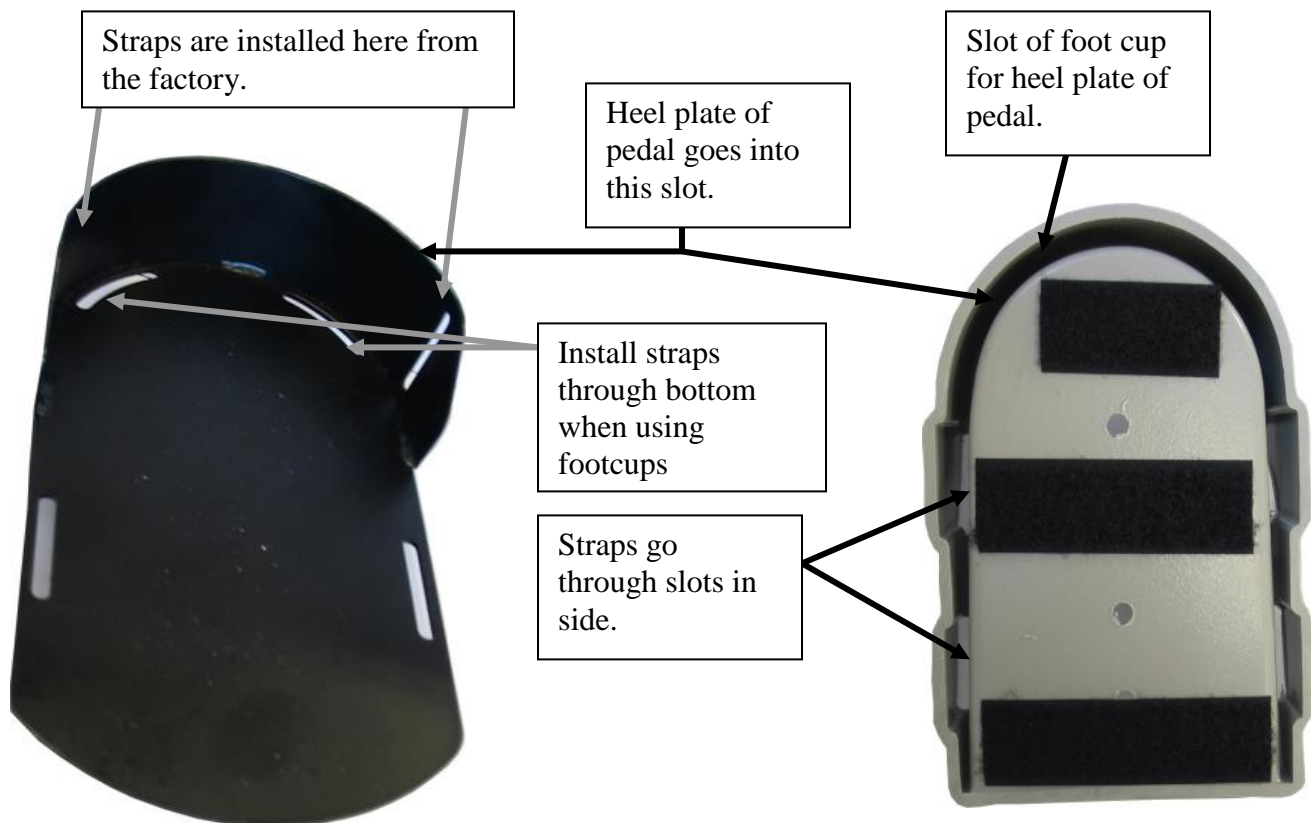

Revise Date: 4/17/2015

n:\amtryke\manuals and instructions\accessories\footcups\foot cups installation.doc