

	ADAPTATION ASSEMBLY INSTRUCTIONS		 	 <i>The therapeutic tricycle for people with disabilities</i> <hr/> <i>For questions or to place an order, please contact:</i> Amtryke, LLC. (800) 838-1845 x114 amtryke@ambucs.org www.ambucs.org
				

AmTryke Snappy Seat #30-04-1500

This seat may be used on the Snappy Tricycle, AM-10 models and AM-12. Notes for 1412 also included



Carton Contents

1-Padded seat back 1-Seat Belt

Tools Required

1-5/32 Allen Wrench

Revised: 4/11/2016


n:\amtryke\manuals and instructions\accessories\snappy seat\snappy seat assembly.docx

1-Padded head rest 1-Metal seat bracket

1-Padded seat

2-Threaded seat knobs

- 1) Attach the padded head rest to the padded back by using four Allen screws and four flat washers as shown. Use a 5/32 Allen wrench to tighten.

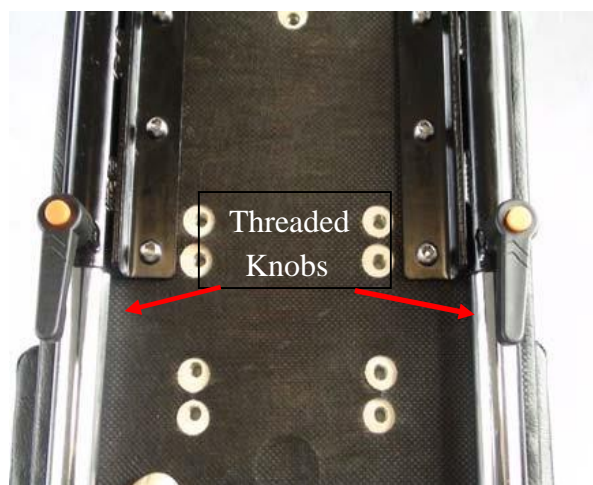

- 2) Attach the metal seat bracket to the back side of the padded back using four Allen screws and four flat washers as shown. Use a 5/32 Allen wrench to tighten.
Note: Two round metal rods point towards the front.

- 3) Slide the padded seat over the metal seat rails as shown.

- 4) Insert the two threaded knobs and tighten.

- 5) Attach the seat belt by using one lag bolt and one wing nut on each side as shown.

Revised: 4/11/2016

n:\amtryke\manuals and instructions\accessories\snappy seat\snappy seat assembly.docx

Step 2



Step 3



6) Attach seat belt center strap with one Allen screw and a flat washer to bottom of padded seat as shown.



7) Located on the bottom of the padded seat are eight predrilled and threaded holes that are used for mounting the Snappy seat to either the Snappy, AM-10 or AM-12.



Pre-Drilled and threaded holes for mounting.


Snappy Seat Parts



Revised: 4/11/2016

n:\amtryke\manuals and instructions\accessories\snappy seat\snappy seat assembly.docx


Bottom of Snappy Seat

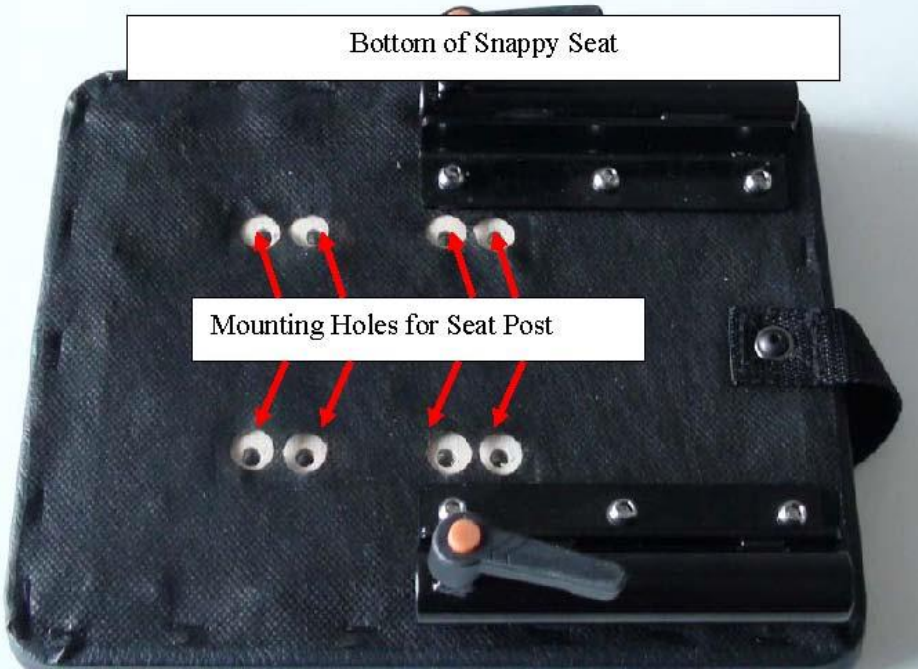

Mounting Holes for Seat Post

Questions or Concerns Regarding your Build? No problem! Please Contact Derek Shaw, AmTryke Technical and Customer Support Coordinator @ 1-800-838-1845 x114 or dereks@ambucs.org

Angled seat post for 1412 to use Snappy Seat

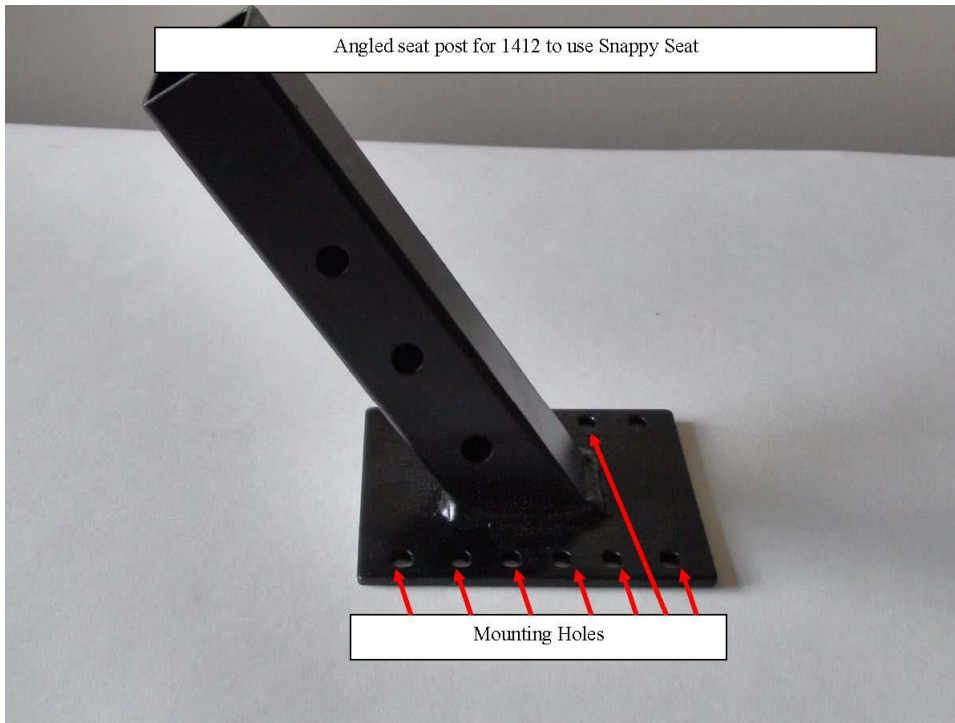

Mounting Holes