



ADAPTATION ASSEMBLY INSTRUCTIONS



AmTryke
The therapeutic tricycle for people with disabilities

For questions or to place an order, please contact:
Amtryke, LLC.
(800) 838-1845 x114
amtryke@ambucs.org
www.ambucs.org




Attach plastic clasp to back of seat using Phillips head screws provided. The cloth will be covering the screw holes and you will need to use a knife or sharp tool to poke a hole through the cloth to expose the screw holes.

Insert top ends of strap through clasp and then push clasp down to secure strap in place.



Revised: 8/2/2016

n:\amtryke\manuals and instructions\accessories\snappy h-harness\snappy seat h-harness.doc

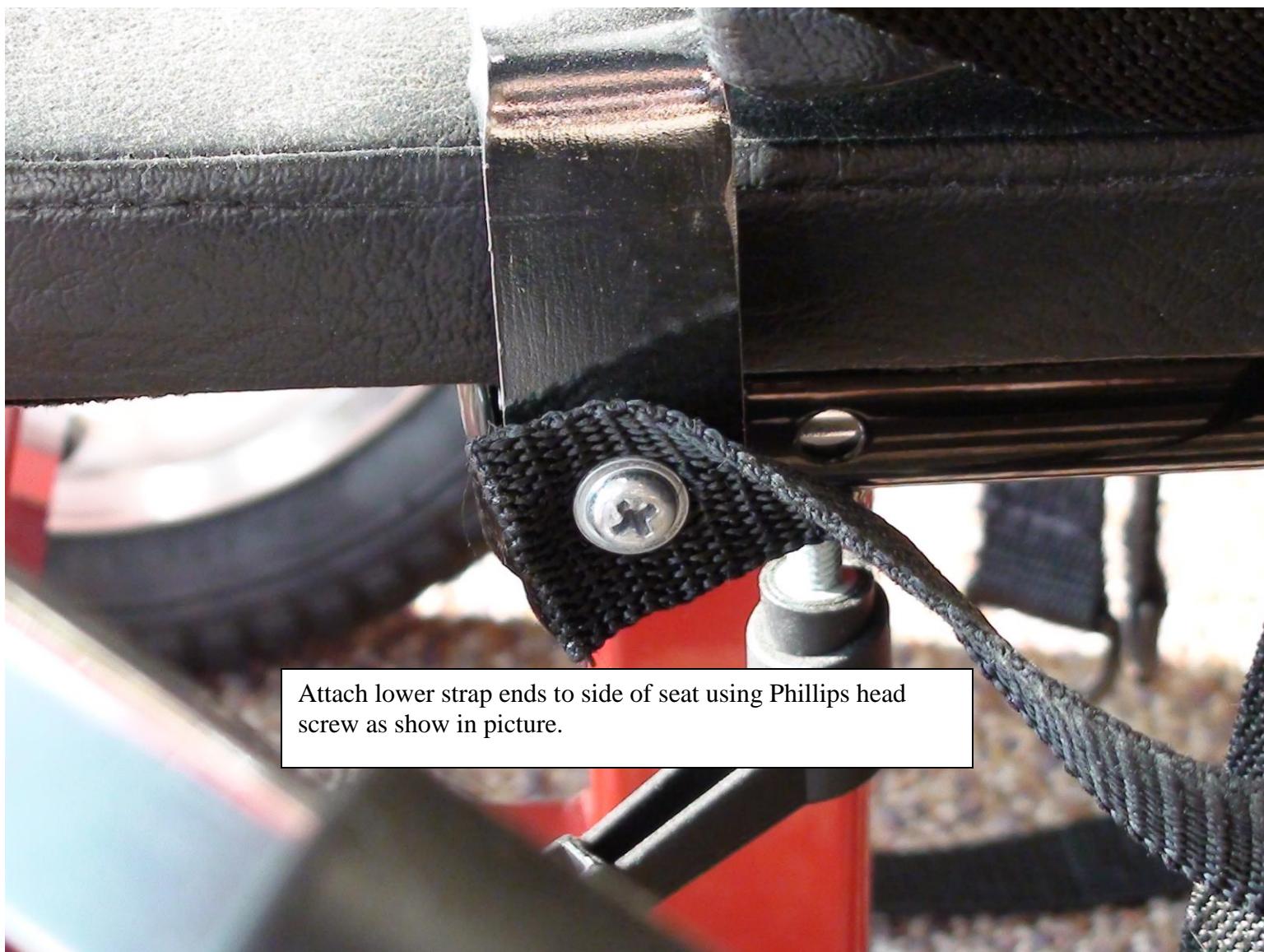

Attach lower strap ends to side of seat using Phillips head screw as show in picture.