

	<h1 style="margin: 0;">ADAPTATION ASSEMBLY INSTRUCTIONS</h1>			<div style="text-align: right;">  </div> <p style="text-align: right; font-size: small;">The therapeutic tricycle for people with disabilities</p> <hr/> <p style="text-align: right; font-size: x-small;">For questions or to place an order, please contact: Amtryke, LLC. (800) 838-1845 x114 amtryke@ambucs.org www.ambucs.org</p>
--	--	---	---	---

**AM/Pro-Series Tractor Seat Bracket
Part # 30-01-0201-AM**




Tools Required

- 14mm wrench or socket
- 15mm wrench or socket

APPLIES TO: AM-12, AM-16, Pro-Series 1416 and Pro-Series 1420

- 1) Unscrew the thumb nut from the quick release, remove the quick release.

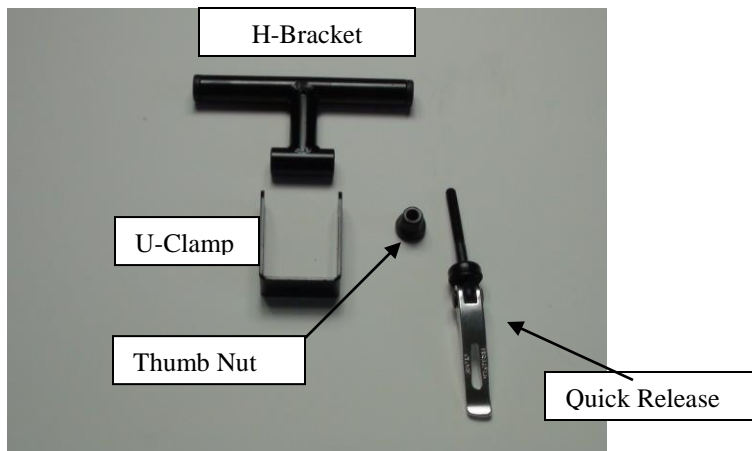

Revised: 6/20/2016

n:\amtryke\manuals and instructions\accessories\tractor seat\tractor seat mounting.doc

- 2) Place the tractor seat on a flat surface so that the underside is exposed.



- 3) Remove one of the nuts that secure the clamp onto the bottom of the seat and remove the clamp.



- 4) Slide one of the long ends of the H-bracket onto the clamp on the bottom of the seat.



- 5) Slide the clamp that you removed over the other long end of the H-bracket, place clamp over threaded stud of tractor seat, re-install the nut by turning clockwise.



6) Place tractor seat on seat post.



7) Slide U-clamp of tractor seat bracket over seat post, Insert quick release shaft through U-bracket and then through H-bracket until it's threaded portion is visible on other side, install thumb nut by turning clockwise, lock cam lever in place.

8) Tighten tractor seat bracket to seat post using a 15mm wrench or socket, tighten tractor seat clamps to bracket using a 14mm wrench or socket.

