

ADAPTATION ASSEMBLY INSTRUCTIONS

AmTryke[®]

The therapeutic tricycle for people with disabilities


For questions or to place an order, please contact:

Amtryke, LLC.
(800) 838-1845 x114
amtryke@ambucs.org
www.ambucs.org

Tractor Seat Mounting to 2700 Series Model AmTryke Part #: 30-01-0201-2700

1. Located on the underside of the tractor seat are two clamps which are attached by nuts.
2. Take the nuts off by turning counter clockwise and remove the clamps.
3. Slide the clamps over the rear brace for the tractor seat (chrome bar shaped like a U)
4. Install the clamps back onto the bottom of the seat and install the two nut by turning clockwise and tighten.
5. Install the tractor seat on the seat post.
6. Located at the rear of the bike are four bolts that secure the rear deck of the bike to its main frame.
7. Remove the nuts off the first set by turning them counter clockwise.
8. Position the seat so that it is level and slide one leg on each side of the seat support bracket (chrome bar) over a bolt, make sure that on each side you use the same hole location.
9. Install the nuts and washer back on by turning clockwise and tighten.

Failure to install this bracket correctly can result in injury to the rider.

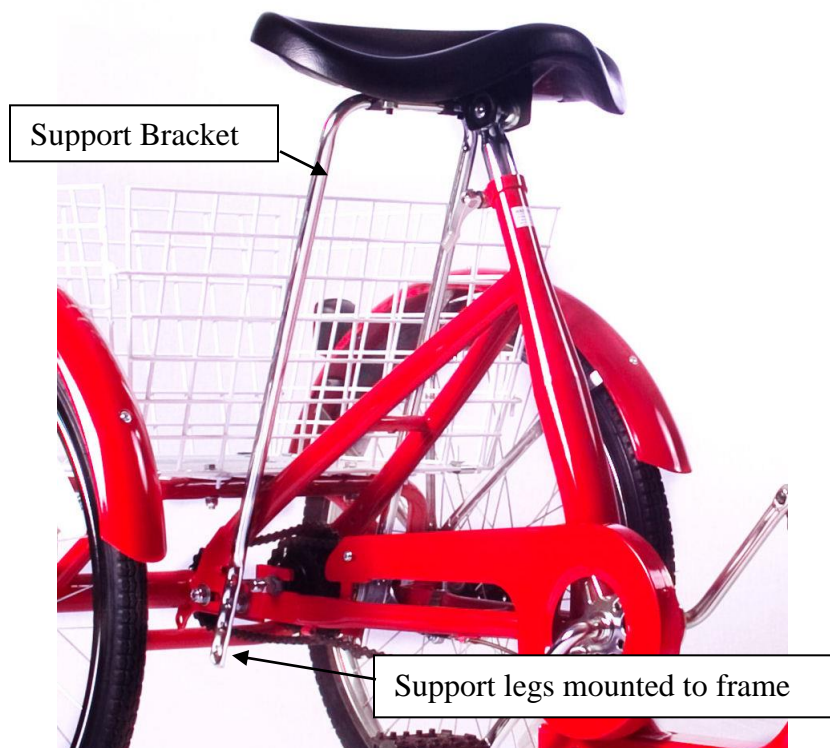

Revised: 6/20/2016

n:\amtryke\manuals and instructions\accessories\tractor seat\tractor seat mounting to 2700.doc